

Pulse Oximeter

Step by Step Guide



1

The settings have been pre-set to present the required readings. There is no requirement to change the settings and this may result in the device not displaying correct readings.

Pulse Oximeter

Step by Step Guide



2

Remove items from box

Pulse Oximeter

Step by Step Guide

3



Turn device over and remove cover

4



Insert batteries

- both batteries need to be face in the same direction as demonstrated above
- negative end of batteries towards perforated edge

Pulse Oximeter

Step by Step Guide



Replace cover



Do not turn device on before finger is inserted

Pulse Oximeter

Step by Step Guide



Insert finger with nail closest to screen & as far towards the back as possible

Pulse Oximeter

Step by Step Guide



Turn device on

Yellow line will move across bottom of screen

Keep finger still and wait for up to 30 seconds

Pulse Oximeter

Step by Step Guide



Reading will be displayed – oxygen levels to the left and pulse rate to the right
Readings can change as live oxygen levels & pulse rate is continuously being monitored

Pulse Oximeter

Step by Step Guide



10

Device will turn off automatically